CTA Carotid 64 Toshiba

Diagnostic Task Detect carotid aneurysms, narrowing or a blockage or arteries Scan mode Helical PosticovLandmark Head or feet first-Supine Topogram AP mA 50 kV 120 Lat 50 mA 120 kV VtyReference mass 120kv 300mA Sure Exp 3D OFF Notation timeptich 0.5/0.641 Detect or Configuration 64x0.5 Table Speed/Increment 20.5 Does reduction na Notwed CTDI ranges* 30mGy-80mGy KR29 Does Notification value 80mGy recon part spacing 1 carotid 3mmx3mm standard 1 carotid thin 0.5mmx.5mm standard 2 ct a carotid thin 0.5mmx.5mm standard 3 coronal MIP 4mmx1mm standard pacs 3 coronal MIP 1mmx1mm standard pacs 3 coronal MIP 2mmx2mm standard pacs 3 coronal MIP 1mmx1mm standard pacs 3 coronal MI		<u> </u>				
Scan mode Helical Prosition/Landmark Head or feet first-Supine toggram AP mA 50 kV 120 Lat 50 mA 120 kV voly/Reterence mass 120kv 300mA Sure Exp 3D OFF Rotation time/pitch 0.5/0.641 Detector Configuration 64x0.5 Table Speedificerement 20.5 Does reduction na Nilewed CTDI ranges* 30mGy-80mGy KR29 Does Notification value 80mGy retical Set body thickness recon part spacing algorithm 1 carotid 3mmx3mm standard pacs 3 coronal MIP 4mmx1mm standard pacs 3 coronal MIP 1mmx1mm standard pacs 5 rt sag oblique MPR 1mmx2mm standard pacs Sean Start/end location 1cm above circle of willis cortact should be injected into RT arm if possible Sean Start woutmetype 60ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist orontrast should be injected into RT arm if possible contrast should be injected into RT arm if pos	Indications	Severe headaches, memory loss, slurred speech, dizziness, blurred or double vision.				
Desition/Landmark Head or feet first-Supine Togogram AP mA 50 kV 120 Lat 50 mA 120 kV Vty/Reference mass 120kv 300mA Sure Exp 3D OFF Notation time/pitch 0.5/0.641 Detector Configuration 64x0.5 Table Speed/increment 20.5 Dese reduction na Nowed CTD/ranges* 30/mGy-80/mGy RE29 Dose Notification value 80/mGy Feloral Set body thickness 1 carotid 3/mx3mm 1 carotid 3/mmx3mm standard 2 cta carotid thin 0.5mmx.5mm standard 2 cta carotid thin 0.5mmx.5mm standard 2 cta carotid thin 0.5mmx.5mm standard 3 coronal MIP 4mmx1mm standard pacs 4 ti sag oblique MPR 1mmx1mm standard pacs 5 rit sag oblique MPR 1mmx1mm standard pacs 5 sag neck MPR 2mmx2mm standard pacs	Diagnostic Task	Detect carotid aneurysms, narrowing or a blockage or arteries				
Topogram AP mA 50 kV 120 Lat 50 mA 120 kV VypReference mass 120kv 300mA Sure Exp 3D OFF Station time/pitch 0.5/0.641 Detector Configuration 64x0.5 fable Specificrement 20.5 Dose reduction na Attempt 30mGy-80mGy KR29 Dose Notification value 80mGy tellcal Set body 1 carotid 3mmx3mm 1 carotid 3mmx3mm 3 coronal MIP 4mmx1mm 4 It sag oblique MPR 1mmx1mm 5 rt sag oblique MPR 1cm below aortic arch 2reov 1cm above circle of willis 0xeov 18cm decrease appropriately V contrast should be injected into RT arm if possible Scan delay manual-start scan when you see blush in ascending aorta NOTE* The Dagrodic Redemans Date (CTD vir) integes between 40 and 30	Scan mode	Helical				
120kv 300mA Sure Exp 3D OFF Stration time/pitch 0.5/0.641 Detector Configuration 64x0.5 Table Specifingration 20.5 Does reduction na Atlewed CTD ranges* 30mGy-80mGy KR29 Does Notification value 80mGy tellical Set body thickness recon part spacing algorithm 1 carotid 3mmx3mm standard parcs 3 coronal MIP 4mmx1mm standard parcs 3 coronal MIP 4mmx1mm standard parcs 3 coronal MIP 4mmx1mm standard pacs 3 coronal MIP 4mmx1mm standard pacs 5 rt sag oblique MPR 1mmx1mm standard pacs Scan Start/end location 1cm below aortic arch 1cm below aortic arch 1cm below aortic arch Scan Start/end location 1cm below acricle of willis 2cortrast should be injected into RT arm if possible Scan delay manual-start scan when you se	Position/Landmark	Head or feet first-Supine				
Relation time/pitch 0.5/0.641 Detector Configuration 64x0.5 Table Speed/Increment 20.5 Does reduction na Nowed CDD ranges* 30mGy-80mGy RR29 Does Notification value 80mGy Helical Set body thickness recon part spacing algorithm 1 carotid thin 0.5mmx.3mm standard none 2 cta carotid thin 0.5mmx.5mm standard pacs 3 coronal MIP 4mmx1mm standard pacs 4 It sag oblique MPR 1mmx1mm standard pacs 5 rt sag oblique MPR 2mmx2mm standard pacs 6 sag neck MPR 2mmx2mm standard pacs Scean Start/end location 1cm above circle of willis pacs form above circle of willis pacs Speov 18cm decrease appropriately v contrast should be injected into RT arm if possible contrast should be injected into RT arm if possible standard mac	Topogram	AP mA 50 kV 120 Lat 50 mA 120 kV				
Detector Configuration 64x0.5 Table Speed/Increment 20.5 Doose reduction na Nowed CTD ranges* 30mGy-80mGy KR29 Dose Notification value 80mGy recon part Spacing algorithm destination 1 carotid thin 0.5 2005 BOMGy telical Set body thickness recon recon part spacing algorithm destination 1 carotid thin 0.5mmx.5mm standard pacs 3 coronal MIP 4mmx1mm standard pacs 5 rt sag oblique MPR 1mmx1mm standard pacs Scan Start/end tocation 1cm above circle of willis spev 18cm decrease appropriately <td c<="" th=""><th>kVp/Reference mass</th><th colspan="4">120kv 300mA Sure Exp 3D OFF</th></td>	<th>kVp/Reference mass</th> <th colspan="4">120kv 300mA Sure Exp 3D OFF</th>	kVp/Reference mass	120kv 300mA Sure Exp 3D OFF			
Table Speed/Increment 20.5 Does reduction na Nitowed CTDI ranges* 30mGy-80mGy KR29 Dose Notification value 80mGy recon part spacing algorithm 1 carotid 1 carotid 2 cta carotid thin 0.5mmx.5mm standard 2 cta carotid thin 3 coronal MIP 4 th sag oblique MPR 4 th sag oblique MPR 5 rt sag oblique MPR 6 sag neck 6 sag neck 7 ontrast volume/type 60ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist contrast volume/type 60ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist contrast should be injected into RT arm if possible scan delay Motte* the Dagoostic Reference Dose (CTD) voly 187mC/velth from CTD phantom, The pastfall limit (ACR and Washington state) te Dam/out-Medi NetoX XR20 Dase Notification Value for an autil bealt all BOm/out-The mastered Mindom/out-The masteret Noterd CTD phan	Rotation time/pitch	0.5/0.641				
Doese reduction na Nowed CTDI ranges* 30mGy-80mGy KR29 Dose Notification value 80mGy telical Set body thickness recon 1 carotid 3mmx3mm standard none 2 cta carotid thin 0.5mmx.5mm standard pacs 3 coronal MIP 4mmx1mm standard pacs 4 it sag oblique MPR 1mmx1mm standard pacs 5 rt sag oblique MPR 1mmx1mm standard pacs 5 rt sag oblique MPR 2mmx2mm standard pacs 5 sag neck MPR 2mmx2mm standard pacs 5 ft sag oblique MPR 1mmx1mm standard pacs standard pacs <th>Detector Configuration</th> <th colspan="4">64x0.5</th>	Detector Configuration	64x0.5				
Allowed CTDI ranges* 30mGy-80mGy KR29 Dose Notification value 80mGy telical Set body thickness recon 1 carotid 3mmx3mm standard none 2 cta carotid thin 0.5mmx.5mm standard pacs 3 coronal MIP 4mmx1mm standard pacs 4 It sag oblique MPR 1mmx1mm standard pacs 5 rt sag oblique MPR 1mmx1mm standard pacs 6 sag neck MPR 2mmx2mm standard pacs 3can Start/end location 1cm below aortic arch 1cm above circle of willis 0 0 0FOV 18cm decrease appropriately v contrast volume/type 60ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist contrast should be injected into RT arm if possible Sean delay manual-start scan when you see blush in ascending aorta NOTE* The Diagnostic Reference Dose (CTDI val) is 75mGywith florm CTDI phanem). The passful limit (ACR and Washington state) a 80mGy. a 80mGy Notif codime head scars on modem s	Table Speed/Increment	20.5				
RR29 Dose Notification value body thickness recon telical Set pecon part spacing algorithm destination 1 carotid 3mmx3mm standard none 2 cta carotid thin 0.5mmx.5mm standard pacs 3 coronal MIP 4mmx1mm standard pacs 4 It sag oblique MPR 1mmx1mm standard pacs 5 rt sag oblique MPR 1mmx1mm standard pacs 6 sag neck MPR 2mmx2mm standard pacs 5 rt sag oblique MPR 1mmx1mm standard pacs 5 rt sag oblique MPR 1mmx1mm standard pacs 6 sag neck MPR 2mmx2mm standard pacs 5 rt sag oblique MPR 1mmx1mm standard pacs 6 sag neck MPR 2mmx2mm standard pacs stant/end location 1cm below aortic arch cm </th <th>Dose reduction</th> <th colspan="4">na</th>	Dose reduction	na				
Hetical Set body thickness recon itelical Set part spacing algorithm destination 1 carotid 3mmx3mm standard none 2 cta carotid thin 0.5mmx.5mm standard pacs 3 coronal MIP 4mmx1mm standard pacs 4 It sag oblique MPR 1mmx1mm standard pacs 5 rt sag oblique MPR 1mmx1mm standard pacs 6 sag neck MPR 2mmx2mm standard pacs Scan Start/end location 1cm below aortic arch 1cm above circle of willis 0 0 OFOV 18cm decrease appropriately v contrast volume/type 60ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist contrast should be injected into RT arm if possible Scan delay manual-start scan when you see blush in ascending aorta NOTE* The Diagnostic Reference Dase (CTD) vol) is 75mGy(with 16cm CTD) plantom). The passifial limit (ACR and Washington state) 1s 80mGy. Most roatine head scane on modern scanners have CTD) vol ranget between 40 and 60mGy. The AAPM r	Allowed CTDI ranges*	30mGy-80mGy				
recon part spacing algorithm destination 1 carotid 3mmx3mm standard none 2 cta carotid thin 0.5mmx.5mm standard pacs 3 coronal MIP 4mmx1mm standard pacs 4 It sag oblique MPR 1mmx1mm standard pacs 5 rt sag oblique MPR 1mmx1mm standard pacs 6 sag neck MPR 2mmx2mm standard pacs Scan Start/end location 1cm below aortic arch 1cm above circle of willis 0 OFOV 18cm decrease appropriately 0 0 0 v contrast volume/type 60ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist contrast should be injected into RT arm if possible scan delay manual-start scan when you see blush in ascending aorta NOTE* The Diagnostic Reference Dose (CTDI vol) is 75mGy(with 16cm CTDI phantom). The pass/tall limit (ACR and Washington state) is 80mGy. Most routine head scans on modem scanners have CTDivol ranges between 40 and 80mGy. The MAPM recommended NEXA XR20 Dose Notification Value for an adult head is 80mGy. The maximum CTDivol should math	XR29 Dose Notification value	80mGy				
1 carotid 3mmx3mm standard none 2 cta carotid thin 0.5mmx.5mm standard pacs 3 coronal MIP 4mmx1mm standard pacs 4 It sag oblique MPR 1mmx1mm standard pacs 5 rt sag oblique MPR 1mmx1mm standard pacs 6 sag neck MPR 2mmx2mm standard pacs Scan Start/end location 1cm below aortic arch 1cm above circle of willis pacs Scan Start/end location 1cm above circle of willis pacs pacs orov 18cm decrease appropriately vontrast volume/type 60ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist contrast volume/type 60ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist contrast should be injected into RT arm if possible standard manual-start scan when you see blush in ascending aorta NOTE* The Diagnostic Reference Dese (CTDI vol) is 75mGy(with 16m CTDI phantom). The passfall limit (ACR and Washington state) is 80mGy. Most routine head acars on modem scanners have CTDivol ranges between 40 and 80mGy. The AAPPM recommended NEXA XR29 Dese Notificatio	Helical Set	body	thickness		recon	
2 cta carotid thin 0.5mmx .5mm standard pacs 3 coronal MIP 4mmx1mm standard pacs 4 It sag oblique MPR 1mmx1mm standard pacs 5 rt sag oblique MPR 1mmx1mm standard pacs 6 sag neck MPR 2mmx2mm standard pacs Scan Start/end location 1cm below aortic arch 1cm above circle of willis 1cm above circle of willis Scan Start/end location 1cm above circle of willis 1cm above circle of willis 1cm above circle of willis Scan of the start start scan when you see blush in ascending aorta contrast should be injected into RT arm if possible 1cm above circle of willis Scan delay manual-start scan when you see blush in ascending aorta NOTE* the biagnostic Reference Dose (CTDI vol) is 75mGy(with 16m CTDI phantom). The pass/fail limit (ACR and Washington state) is 80mGy. Most routine head scans on modern scanners have CTDIvol ranges between 40 and 60mGy. The AAPM recommended NEXA XR29 Dose Notification Value for an aduit head is 80mQy. The maximum CTDivol should match The AAPM recommended NEXA XR29 Dose Notification Value for an aduit head is 80mQy. The maximum CTDivol should match		recon part	spacing	algorithm	destination	
3 coronal MIP 4mmx1mm standard pacs 4 It sag oblique MPR 1mmx1mm standard pacs 5 rt sag oblique MPR 1mmx1mm standard pacs 6 sag neck MPR 2mmx2mm standard pacs Scan Start/end location 1cm below aortic arch 1cm below aortic arch 1cm above circle of willis orov 18cm decrease appropriately 00ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist ocntrast volume/type 60ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist scan delay manual-start scan when you see blush in ascending aorta NOTE* The Diagnostic Reference Dose (CTDI vol) is 75mGy(with 16m CTDI phantom). The passifal limit (ACR and Washington state) is 80mGy. Mest routine head scans on modern scanners have CTDIvol ranges between 40 and 60mGy. The AAPM recommended NEXA XR20 Dose Notification Value for an adult head is 80mGy. The maximum CTDIvol should match		1 carotid	3mmx3mm	standard	none	
4 It sag oblique MPR 1mmx1mm standard pacs 5 rt sag oblique MPR 1mmx1mm standard pacs 6 sag neck MPR 2mmx2mm standard pacs Scan Start/end location 1cm below aortic arch 1cm below aortic arch 1cm above circle of willis oprov 18cm decrease appropriately 18cm decrease appropriately 18cm decrease appropriately v contrast volume/type 60ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist contrast should be injected into RT arm if possible scan delay manual-start scan when you see blush in ascending aorta NOTE* The Diagnostic Reference Dose (CTDI vol) is 75mGy(with 16cm CTDI phantom). The pass/fail limit (ACR and Washington state) is 80mGy. Most routine head scans on modern scanners have CTDI vol and 80mGy. The maximum CTDIvol should match		2 cta carotid thin	0.5mmx .5mm	standard	pacs	
5 rt sag oblique MPR 1mmx1mm standard pacs 6 sag neck MPR 2mmx2mm standard pacs Scan Start/end location 1cm below aortic arch 1cm above circle of willis 1cm above circle of willis DFOV 18cm decrease appropriately 00ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist v contrast volume/type 60ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist contrast should be injected into RT arm if possible manual-start scan when you see blush in ascending aorta NOTE* The Diagnostic Reference Dose (CTDI vol) is 75mGy(with 16cm CTDI phantom). The pass/fail limit (ACR and Washington state) is 80mGy. Most routine head scans on modern scanners have CTDIvol ranges between 40 and 60mGy. The AAPM recommended NEXA XR29 Dose Notification Value for an adult head is 80mGy. The maximum CTDIvol should match		3 coronal MIP	4mmx1mm	standard	pacs	
6 sag neck MPR 2mmx2mm standard pacs Scan Start/end location 1cm below aortic arch 1cm above circle of willis DFOV 1cm above circle of willis v contrast volume/type 60ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist contrast should be injected into RT arm if possible contrast should be injected into RT arm if possible Scan delay manual-start scan when you see blush in ascending aorta NOTE* The Diagnostic Reference Dose (CTDI vol) is 75mGy(with 16cm CTDI phantom). The pass/fail limit (ACR and Washington state) is 80mGy. Most routine head scans on modern scanners have CTDIvol ranges between 40 and 60mGy. The AAPM recommended NEXA XR29 Dose Notification Value for an adult head is 80mGy. The maximum CTDIvol should match		4 It sag oblique MPR	1mmx1mm	standard	pacs	
Scan Start/end location 1cm below aortic arch 1cm above circle of willis 0FOV 18cm decrease appropriately V contrast volume/type 60ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist contrast volume/type 60ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist contrast should be injected into RT arm if possible Scan delay manual-start scan when you see blush in ascending aorta NOTE* The Diagnostic Reference Dose (CTDI vol) is 75mGy(with 16cm CTDI phantom). The pass/fail limit (ACR and Washington state) is 80mGy. Most routine head scans on modern scanners have CTDIvol ranges between 40 and 60mGy. "The AAPM recommended NEXA XR29 Dose Notification Value for an adult head is 80mGy. The maximum CTDIvol should match		5 rt sag oblique MPR	1mmx1mm	standard	pacs	
1cm above circle of willis DFOV 18cm decrease appropriately V contrast volume/type 60ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist contrast should be injected into RT arm if possible Scan delay manual-start scan when you see blush in ascending aorta NOTE* The Diagnostic Reference Dose (CTDI vol) is 75mGy(with 16cm CTDI phantom). The pass/fail limit (ACR and Washington state) is 80mGy. Most routine head scans on modern scanners have CTDI vol ranges between 40 and 60mGy. "The AAPM recommended NEXA XR29 Dose Notification Value for an adult head is 80mGy. The maximum CTDIvol should match		6 sag neck MPR	2mmx2mm	standard	pacs	
DFOV 18cm decrease appropriately V contrast volume/type 60ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist contrast should be injected into RT arm if possible contrast should be injected into RT arm if possible Scan delay manual-start scan when you see blush in ascending aorta NOTE* The Diagnostic Reference Dose (CTDI vol) is 75mGy(with 16cm CTDI phantom). The pass/fail limit (ACR and Washington state) is 80mGy. Most routine head scans on modern scanners have CTDI vol ranges between 40 and 60mGy. "The AAPM recommended NEXA XR29 Dose Notification Value for an adult head is 80mGy. The maximum CTDI vol should match	Scan Start/end location	1cm below aortic arch				
V contrast volume/type 60ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist contrast should be injected into RT arm if possible Scan delay manual-start scan when you see blush in ascending aorta NOTE* The Diagnostic Reference Dose (CTDI vol) is 75mGy(with 16cm CTDI phantom). The pass/fail limit (ACR and Washington state) is 80mGy. Most routine head scans on modern scanners have CTDI vol ranges between 40 and 60mGy. "The AAPM recommended NEXA XR29 Dose Notification Value for an adult head is 80mGy. The maximum CTDIvol should match		1cm above circle of willis				
Scan delay Contrast should be injected into RT arm if possible Scan delay manual-start scan when you see blush in ascending aorta NOTE* The Diagnostic Reference Dose (CTDI vol) is 75mGy(with 16cm CTDI phantom). The pass/fail limit (ACR and Washington state) is 80mGy. Most routine head scans on modern scanners have CTDI vol ranges between 40 and 60mGy. *The AAPM recommended NEXA XR29 Dose Notification Value for an adult head is 80mGy. The maximum CTDI vol should match	DFOV	18cm decrease appropriately				
Scan delay manual-start scan when you see blush in ascending aorta NOTE* The Diagnostic Reference Dose (CTDI vol) is 75mGy(with 16cm CTDI phantom). The pass/fail limit (ACR and Washington state) is 80mGy. Most routine head scans on modern scanners have CTDI vol ranges between 40 and 60mGy. *The AAPM recommended NEXA XR29 Dose Notification Value for an adult head is 80mGy. The maximum CTDI vol should match	IV contrast volume/type	60ml isovue 370 3-4cc/sec Performed as directed by the supervising radiologist				
NOTE* The Diagnostic Reference Dose (CTDI vol) is 75mGy(with 16cm CTDI phantom). The pass/fail limit (ACR and Washington state) is 80mGy. Most routine head scans on modern scanners have CTDI vol ranges between 40 and 60mGy. *The AAPM recommended NEXA XR29 Dose Notification Value for an adult head is 80mGy. The maximum CTDI vol should match		contrast should be injected into RT arm if possible				
is 80mGy. Most routine head scans on modern scanners have CTDIvol ranges between 40 and 60mGy. *The AAPM recommended NEXA XR29 Dose Notification Value for an adult head is 80mGy. The maximum CTDIvol should match	Scan delay	manual-start scan when you see blush in ascending aorta				
is 80mGy. Most routine head scans on modern scanners have CTDIvol ranges between 40 and 60mGy. *The AAPM recommended NEXA XR29 Dose Notification Value for an adult head is 80mGy. The maximum CTDIvol should match	NOTE*	The Diagnostic Reference Dose (CTDI vol) is 75mGy(with 16cm CTDI phantom). The pass/fail limit (ACR and Washington state)				
the dose notification value. Exams with CTDI vol values less than the minimum allowed range should not be performed unless						
		the dose notification value. Exams with CTI	DI vol values less than the minimum	allowed range should not be performed unless		

bolus track at arch, trigger is + ?

at arch, at arch, t